# Steam Inhaler Inhalador de Vapor Inhalateur Vapeur

USE AND CARE MANUAL  
MANUAL DE USO Y CUIDADO  
MANUEL D'UTILISATION ET D'ENTRETIEN

This steam inhaler enables the user to breathe in warm water vapor for better breathing and a more comfortable sleep. If you have any questions about the operation of your Vicks Steam Inhaler, call our toll-free Consumer Relations line at 1-800-VAPOR-1-2 or e-mail us at [Consumerrelations@kaz.com](mailto:Consumerrelations@kaz.com).

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**Model V1300N**  
**Modelo V1300N**  
**Modèle V1300N**

Congratulations on your purchase of the **Vicks Steam Inhaler**. Your new Inhaler provides all the benefits of vapor therapy at home, at the office or when traveling.

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## Directions for Use

Rinse unit out thoroughly before first use.

1. Remove top of unit from base by twisting in opposite directions and lifting off top.
2. Fill base to water line with hot (not boiling) tap water. NOTE: If you overfill, the unit will leak.
3. Open Vicks VapoPad® packet and add to the water. As an alternative, one teaspoonful of Vicks VapoSteam® or Kaz Inhalant can also be used. Replace the top.

4. Breathe in the soothing Vicks vapors.  
CAUTION: Product contains hot water. To avoid hot water spilling after filling, be sure to properly secure top housing by twisting completely so it snaps back into place

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## Care and Cleaning

After each use, thoroughly wash and dry your inhaler. The unit is top shelf dishwasher safe.

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## Use with Vicks VapoPad® or Vicks VapoSteam®

Your inhaler was designed to work with Vicks VapoPad®, Vicks VapoSteam® or Kaz Inhalant. The menthol and eucalyptus vapors from these provide soothing comfort.

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## Recommended Usage for the Vicks Menthol VapoPad (VSP-19/VSP-19-CAN):

- Not recommended for infants under 10 lbs (4.5 kgs)
  - For children between 10 – 22 lbs (4.5-10 kgs) do not use more than 2 pads in a 24 hour period
- VapoPad Ingredients: Eucalyptus Oil, Menthol, Glycol, Cedar Leaf Oil & Others

**⚠ WARNING:** Do not rub face or eyes after touching the VapoPad. Oils in the pad could act as an irritant to eyes.

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## One Year Limited Warranty

Your Vicks Steam Inhaler is warranted against defective materials or workmanship for one year from date of purchase. Any defective part will be replaced at no charge if it has not been tampered with and the appliance has been used according to our printed directions. This warranty gives you specific legal rights, as well as other rights which may vary from state to state and from province to province.

If you experience unsatisfactory operation, please call the Kaz Consumer Service Department at 1 (800) 477-0457, or write to Kaz USA, Inc. Consumer Service Dept. 250 Turnpike Road, Southborough, MA 01772, explaining just how the inhaler is operating. Our Consumer Service Representative will advise you how to correct the problem yourself, or will ask you to return the Inhaler for repair or replacement.

Please go to [www.kaz.com](http://www.kaz.com) and register your product under the Customer Care Center and receive product information updates and new promotional offers.

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*Manufacturing Quality Healthcare Products for Over 75 Years*

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Kaz USA, Inc. • 250 Turnpike Road, Southborough, MA, USA 01772 • [www.kaz.com/vicks](http://www.kaz.com/vicks)

Distributed by / Distribuido por / Distribué par Kaz Canada • 510 Bronte St. South, Milton, ON L9T 2X6 • [www.kaz.com/vicks](http://www.kaz.com/vicks)

This product is manufactured by Kaz USA, Inc. under license from The Procter & Gamble Company.

Vicks and other associated trademarks are owned by The Procter & Gamble Company.

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© 2013 All Rights Reserved/Todos los Derechos Reservados/Tous droits réservés, Kaz USA, Inc.
Este producto es fabricado por Kaz USA, Inc. bajo licencia de The Procter & Gamble Company.
Ce produit est fabriqué pour Kaz Canada, Inc. sous licence de la société Procter & Gamble.