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**ISOTONIC SALINE SPRAY / Medical device**

Saltwater washes (saline lavage or irrigation) help keep the nasal passages open by washing out thick or dried mucus. They can also help improve the function of cilia that help clear the sinuses. This can help prevent the spread of infection to the other sinuses and reduce post-nasal drip. It also can make the nose feel more comfortable by keeping the mucous membranes moist.

**PRESENTATION:**

Spray with isotonic saline solution.

PRODUCT CODE	PRESENTATION FORM / QUANTITY	NO. OF PIECES/ INDIVIDUAL BOX
2SNBL	Spray with 100 ml isotonic saline solution	1

#### COMPOSITION:

Purified water, sea salt. 1 ml of solution contains 9 mg of sea salt.

#### CLINICAL BENEFITS:

- Nasal prophylactic hygiene without adverse effects
- Improves the natural protection of the nasal mucosa

#### PERFORMANCE CHARACTERISTICS:

- Saline solution used for nasal rinsing; highly effective in improving allergy symptoms and shortening the duration of a sinus infection
- Use Bag-on-Valve technology: isotonic saline solution is filled in a bag contained within the can. The propellant gas is filled into the area between the aluminum can and the bag containing the solution. Isotonic saline solution is therefore separated from the propellant gas so that when the container is dispensed, only the solution is expelled, while the gas remains in the package and is never mixed with the product; thus:
  - The product's life span increases without adding preservatives.
  - The solution is well protected from external contamination

#### INDICATIONS:

- Products are recommended for nasal prophylactic hygiene. The isotonic saline solution can be used for newborns and children.

#### CONTRA-INDICATIONS:

- Do not use if nose or ears are completely plugged or in the event of ear infection
- Do not swallow

#### INSTRUCTIONS FOR USE:

- Gently insert the spray in the nose/ear
- Press the spray briefly. If your nose is clogged, continue to press the sprayer few seconds
- Allow to drain the excess solution in order to extract the mucus, and then blow
- Repeat the process in the other nostril to ensure the complete removal of mucus
- Spray and wash several times a day, depending of the severity of symptoms

#### PRECAUTIONS FOR USE:

- Be sure to use the spray at room temperature: 20°C – 25°C
- Do not apply when a child's head is back in order to prevent fluid from going down the throat
- Do not completely block or occlude nasal passage during use to avoid pressure in the nose or ears
- Consult a healthcare practitioner if symptoms persist or worsen, or if there is no improvement after you have tried to clear the nose; if the discharge is persistently thick and yellow or green; if you have any other symptoms of illness such as fever or ear infection
- For single patient use only; do not use the product at the same time in your ears and nose

#### ADDITIONAL PRECAUTIONS:

- The spray with isotonic saline solution is only for use in the nose. Do not spray near eyes or mouth
- Before each use, is recommended to clean the spray nozzle with soap and water and dry it
- Storage:
  - Protected from humidity and sunshine, in an upright position
  - Temperature: max. 35° C.

#### UNDESIRABLE SIDE EFFECTS:

- A temporary tingling sensation if the nasal mucus membrane is highly irritated.

#### WARNINGS:

- The product is exclusively for external use; ingestion of the solution may cause stomach problems
- For people with mental disabilities, the products should only be used under supervision of a mentally healthy person
- Do not use the product if the packaging is damaged
- The products should not be further used after the expiration date
- The products should be kept away from children and pets. Penetration in the ear or in the nose of hair or stains from pets can cause nose/ear infection. Also, hair or stains from pets can be swallowed and causes serious illnesses
- Keep away from heat / sparks / open flames or hot surfaces
- Pressurized container. Do not pierce or burn, even after use
- Do not expose to temperatures above 50 °C.

#### PACKAGING:

Spray with 100 ml isotonic saline solution contains about 300 spray doses.

**NOTE:** Any serious incident that has occurred in relation to TAISSIS CONCEPT S.R.L. products should be reported to TAISSIS CONCEPT S.R.L. and the competent authority of the Member State in which the user and/or patient is established

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[www.nasolibero.com](http://www.nasolibero.com)

Regulatory status: IIa Class medical devices according to MDD 93/42/EEC

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