GET READY!

babyono

<b>GB</b>	3-in-1 Electric breast pump	2
<b>PL</b>	Laktator elektryczny 3 w 1	4
<b>BG</b>	Електрическа помпа за кърма 3 в 1	6
<b>CZ</b>	Elektrická odsávačka 3v1	8
<b>DE</b>	Elektrische muttermilchpumpe 3 in 1	10
<b>ES</b>	Extractor de leche materna eléctrico 3en1	12
<b>FR</b>	Tire-lait électrique 3en1	14
<b>GR</b>	Ηλεκτρική συσκευή άντλησης μητρικού γαλακτός 3 σε 1	16
<b>HU</b>	3 az 1-ben Elektromos mellszívó	18
<b>LT</b>	Elektrinis pientraukis 3 viename	20
<b>LV</b>	Elektriskais krūts piena sūknis trīs vienā	22
<b>NL</b>	Elektrische borstkolf 3in1	24
<b>RO</b>	Pompă de sân electrică 3 în 1	26
<b>RU</b>	Молокоотсос электрический 3в1	28
<b>SK</b>	Elektrická odsávačka 3v1	30
<b>SL</b>	Električni laktator 3 v 1	32
<b>UA</b>	Молоковідсмоктувач електричний 3в1	34
<b>XS</b>	Električna pumpa za izmuzivanje majčina mleka - laktator 3 u 1	36

# 3-in-1 Electric breast pump

- Three-phase program comprising 10 suction power modes
  - massage to stimulate suction
  - massage to stimulate suction along with breast milk expressing
  - breast milk expressing
- discreet and comfortable to use in travel
- powered with rechargeable batteries
- possibility to feed the baby right after pumping

## NOTE

Before the first use of the breast pump, please read carefully the operation manual and the warnings contained therein. Please keep the instruction manual for future reference.

- Please use the breast pump in compliance with its intended use described in the manual.
- Do not use the breast pump if it has been physically damaged.
- Do not place the breast pump in the vicinity of any devices that generate powerful magnetic fields.
- Do not expose the breast pump to direct sunlight.
- Keep it out of reach of children.
- Please keep the pumping unit in a dry place and do not immerse it in water.
- Before use, check all the elements and make sure they have not been damaged.
- Keep the breast pump clean. The feeding bottle, the silicone breast cup, the breast pump body and the silicone valves may be sterilised.
- Make sure there is no liquid in the silicone tube. Do not use the breast pump if any liquid is visible in the tube. It has to be dried first.

## LIST OF PARTS

1. silicone breast cup
2. breast pump body
3. breast shield tunnel
4. silicone membrane
5. pump body cap
6. silicone tubing
7. pumping unit
8. feeding bottle
9. charger
10. teat
11. bottle cap
12. cap
13. silicone valve
14. bottle sealing disc

## ASSEMBLY INSTRUCTION

1. Insert the breast shield tunnel (3) into the breast pump body (2).
2. Put the silicone breast cup (1) on the breast shield tunnel.
3. Put the silicone membrane (4) on the top of the breast pump body. Then put the pump body cap (5) over the membrane, turning it to the right until it blocks.
4. Push the silicone valve (13) into the bottom of the body (make sure it's been done correctly).
5. Fix the feeding bottle (8).
6. Put the silicone tubing (6) onto the aperture in the body, and connect the other end of the tube to the pumping unit (7).

## BATTERY CHARGING

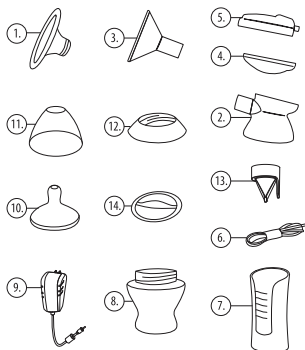
**NOTE!** The battery must be fully charged before first use. The breast pump is equipped with a lithium-ion battery, so it does not require the use of AA type batteries or permanent connection to power supply. It is recommended to charge the breast pump before starting to express milk. The battery provided with the breast pump can be charged approx. 300 times. Stop charging when the battery is full in order to avoid harming the battery. In order to extend the life of the breast pump and reduce the electricity consumption by the battery it should not be charged for more than 6 hours.

The breast pump is equipped with a battery. In order to charge it, connect the breast pump to the charger and then to the power outlet. Charging will take about 100 minutes. When the battery is fully charged, all the blue LEDs will light up. The breast pump can be charged at any time with no need to wait for the battery to discharge.

After the battery is fully charged, the breast pump will operate for approx. 100 minutes. **NOTE!** The breast pump has a timer. After 45 minutes of continuous operation, it switches off automatically. To resume the operation, press the power button. Shortly before the battery is discharged, the power LED (first on the left) will start flashing.

## OPERATION MANUAL

1. Please bear in mind that stress has a tremendous effect on the amount of breast milk produced. Before you start pumping, make sure to eliminate any stressful situations. Try to relax and do not

forget about hygiene. Wash your hands with water and soap, and your breast with water only.

2. Many women feel increased thirst during breast feeding or breast milk expressing. Have something to drink at hand so that you don't have to interrupt the pumping process.
3. Do not feel discouraged in case you experience any problems at the beginning – in a short time you will work it out and breast milk pumping will no longer be a problem for you.
4. Sit down comfortably and take a position in which you will be able to lean slightly forward.
5. Make sure the breast pump batteries have been charged.
6. Take the sterilised breast pump body and make sure all its elements have been put in correctly.
7. Put the breast pump shield onto your breast and turn on the device (  )
8. The (  ) button provides the suction stimulation, i.e. massage, via short, quick movements.
9. Then you can choose one of the two pumping programs:
  - The first program comprises suction stimulation along with milk expressing (  +  ). Expressing takes place after 5 suction stimulating movements. By pressing the same button consecutively you will increase the suction power. The blue LED will then change its position to another. You can choose from 5 suction power modes
  - The second mode is breast milk expressing (  ). This program does nothing else but milk expressing. Each consecutive pressing of the button will increase the suction power. In the case of this program you also have 5 suction power modes to choose from.
10. If you don't feel any suction, make sure air has not got in between your breast and the silicone breast cup. If so, take your breast out and put it back in the breast shield tunnel.
11. When milk has stopped flowing out, finish the pumping.
12. Amounts of expressed milk will vary, as they depend on the level of the lactation, time of day and the amounts of liquids you drink. When your breasts are full, milk pumping may even take 30 minutes.
13. After finishing the pumping, disconnect the silicone tubing from the pump, wash the other elements of the body and prepare them for sterilisation. NOTE: DO NOT IMMERSE THE PUMPING UNIT IN WATER. IF YOU HAVE WASHED THE SILICONE TUBING, DRY IT THOROUGHLY BEFORE USING.

#### BREAST PUMP CLEANING

Disassemble all the elements of the breast pump in the reverse order in relation to the assembly instruction. Before the first and each further use, all the elements of the breast pump body should be thoroughly washed in warm water with baby soap, and rinsed several times. The breast pump body, breast shield tunnel and the feeding bottle may be put in a dishwasher (the upper shelf). However, please bear in mind that they should be removed from the dishwasher immediately after finishing the rinsing cycle and dried, they may not be heated up. This could lead to deforming and permanent damage of the elements. The elements of the body that come into contact with milk may be sterilised in BabyOno steam sterilization bags.

Never use any disinfectants to clean them. Store the product in a warm and dry place. Do not expose the product to direct sunlight or put it close to other hot places.

#### NATURAL NURSING FEEDING BOTTLE AND TEAT

The enclosed teat is intended for feeding bottles from the Natural Nursing product family. Before each use, please check the teat quality by stretching

it in different directions. If the teat is damaged, it should be immediately replaced with a new one. Do not enlarge the apertures in the teat due to the risk of damaging the silicone structure. Due to hygiene considerations it is recommended to replace the teat with a new one once in every 1 – 2 months.

#### IMPORTANT FOR YOUR BABY'S HEALTH AND SAFETY

- Never leave your child unattended during feeding.
- Never use feeding teats as soothers.
- Continuous and prolonged sucking of liquids may cause tooth decay.
- Always check the food temperature before feeding.
- Please remember that the sterilized bottle elements must be assembled with clean hands.

The product complies with the requirements of the EN 14350 standard.

#### EXPRESSED BREAST MILK STORAGE

IMPORTANT: Please make sure any expressed breast milk is correctly stored! Incorrectly stored breast milk loses its nutritive value and may even be harmful to your baby.

We recommend BabyOno sterile bags for breast milk storing and freezing.

STORAGE PLACE	STORAGE TIME	
	FOR HEALTHY BABIES	FOR PREMATURE AND ILL BABIES
In room temperature	8 – 12 hours	1 – 4 hours
In a fridge ( 5 °C )	3 – 4 days	24 – 48 hours
In a freezer compartment ( -10 °C )	2 weeks	1 week
In a freezer ( -18 °C )	6 – 12 months	3 – 6 months

#### NOTE

- Any thawed breast milk must not be frozen again. Any unused milk should be discarded.
- Do not mix any thawed breast milk with freshly pumped breast milk.
- Please freeze only fresh breast milk.
- Breast milk should be defrosted slowly. To do this, frozen milk should be placed in a refrigerator until it takes a liquid form.
- To warm the breast milk, it is recommended to use a water bath with a temperature not exceeding 50 °C.
- To preserve its nutritive value, breast milk must not be boiled, heated up directly on a cooker or in a microwave!
- Ready to serve breast milk should have the human body temperature, i.e. no more than 37 °C.

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