
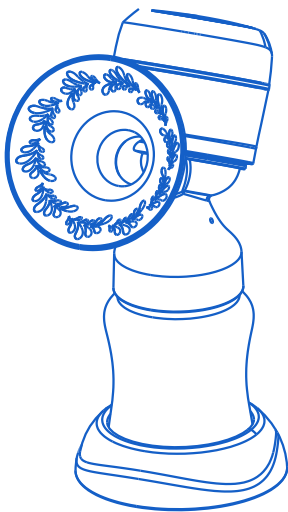
LEIA

integrated LED breast pump


INSTRUCTIONS FOR USE

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
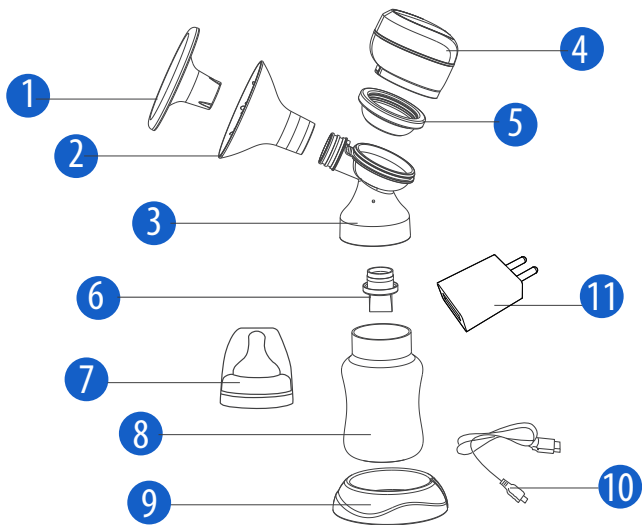
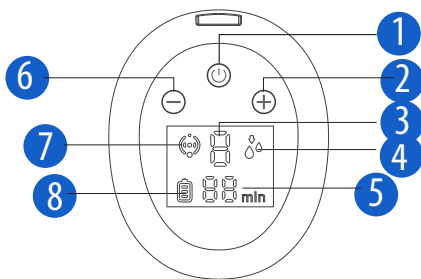
II.

III.

Freshly Expressed Breast Milk Storage Guidelines (for Healthy Term Babies) ¹			
² Room Temperature	³ Cooler With Ice Pack	⁴ Refrigerator	⁵ Freezer
⁶ 4-6 hours at 66-78°F (19-26°C)	⁷ 24 hours at 59°F (15°C)	⁸ 3-8 days at 39°F or lower (4°C)	⁹ 6-12 months 0 - -4°F (-18 - -20°C)

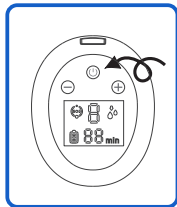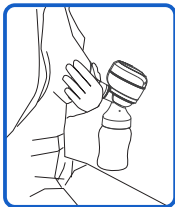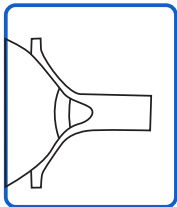IV.

V.

INTRODUCTION

Congratulations on the birth of your baby! You will soon find that breastfeeding your baby is a worthwhile experience. Breast milk will provide your baby with the best nutrition for healthy development.

This electric breast pump can simulate the baby's sucking actions to provide comfortable and pain-free service for breastfeeding mothers. FOR PROPER AND SAFE USE OF THE PUMP, PLEASE READ THE INSTRUCTIONS CAREFULLY AND PLACE IT TO BE EASILY ACCESSIBLE FOR FUTURE REFERENCE.

PRODUCT FEATURES

- Two modes with 9 suction levels: Massage and Pumping, where it can be adjusted to 9 levels of drainage mode and 6 levels of massage mode, depending on the needs of the mother.
- Hygienic & Prevents Leaking Anti-back flow protection which separates the milk and air, ensures milk's hygiene. Soft silicone breast shield which perfectly fit the shape of mom's breasts, can effectively prevent milk leakage.
- Digital Display & Timer: Digital display features a memory button, timer and backlight for easy tracking and recording pump sessions. Moms are Allowed to save favorite pumping pattern.
- BPA FREE Material: All material in the pump and accessories are BPA-free Material
- Super Quiet: The noise of motor is controlled under 55dB. It is a very soft sounds which will provide a good sleeping environment for your baby and family.
- Type-C USB Charge: Using large capacity durable lithium battery, fully charged can be used for 2 hours, no need to connect to AC power. Moms can enjoy pumping anytime anywhere.

I. PRODUCT COMPONENTS

1. Dust Cap; 2. Silicone Shield; 3. Connector Body; 4. Pump Machine; 5. Silicone diaphragm; 6. Valve; 7. Milk Bottle; 8. Bottle Base; 9. Bottle Cap Set; 10. Charging Cable; 11. Adaptor

Technical Specification: Rated Voltage: DC 5V Rated Power: 5W; Battery Type: Lithium Battery Capacity: 1400mah

II. DISPLAY SPECIFICATION

1. Power Button/Mode change; 2. Suction Increase; 3. Massage/SuctionLevel; 4. Suction Mode; 5. Session Duration; 6. Battery Percent; 7. Massage Mode; 8. Suction decrease;

Power Button: Press 2 seconds to start or off the breast pump.

Mode Change: Touch the screen to change mode, do not touch long time. 2 minutes massage mode is initial set-up and the mode will be as same as the last time you used. It will change to pump mode automatically and will stop after 28 minutes.

Suction Level: + suction increase - suction decrease

(Massage: L 1-L6 Pumping: L 1-L9)

Battery Percent: it shows the rest battery percent, please charge when is low battery

Session Duration: It shows the time you have used for the mode you chose; the machine will stop every 30 minutes.

III. ASSEMBLY INSTRUCTION

Step 1: Press the suction bowl and pump machine into the connector body.

Step 2: Push the silicone shield onto the connector body.

Step 3: Put the valve into the connector body.

Step 4: Fasten the milk bottle onto the connector body.

Notes:

- Wash hands thoroughly with soap and water before touching breast pump, accessories and breasts.
- Check breast pump kit components for wear or damage before use. Replace if necessary.
- Always inspect all parts prior to use for cleanliness.

IV. STORING BREAST MILK

- As soon as you are finished pumping, place the breast milk bottle or breast milk storage bag in the cooling bag with ice pack when you are outside.
- Transfer breast milk bottles or breast milk storage bags to the refrigerator or freezer once you are home.

1. FRESHLY EXPRESSED BREASTMILK STORAGE GUIDELINES (FOR HEALTHY TERM BABIES).

2. Room temperature; 3. Cooler with ice pack; 4. Refrigerator; 5. Freezer; 6. 4-6 hours at 19-26°C (66-78°F); 7. 24 hours at 15°C (59°F); 8. 3-8 days at 4°C (39°F); 9. 6-12 months at 0- -4°C (-18- -20°F);

V. OPERATE INSTRUCTION

Step 1- Place the breast shields on your breasts so that your nipples are properly centered in the tunnels.

Step 2- Hold the breast shield and connector onto your breast with your thumb and index finger. Support your breast with the palm of your hand.

Step 3- Turn on the breast pump by pressing the power button and choose the mode you prefer.

Step 4- Turn off the breast pump by pressing the power button after using and make the dust cap covered.

Notes:

1. Your breast pump will begin in Stimulation Phase 0. Adjust the pumping level using the (-)/(+) controls to find a comfortable level.

Stimulation Phase-a fast sucking pump rhythm to stimulate milk flow

2. After two minutes, your pump will automatically transition to Expression Phase b. Adjust the pumping level using the (-)/(+) controls to find your maximum confirmed Vacuum. If your milk begins to flow before two minutes, press the mode change button POWER

To manually transition to Expression Phase

Expression Phase-a slower sucking/pumping rhythm for gentle and efficient milk removal after milk has started flowing.

3. Once you are pumping in the Expression Phase, increase the speed/vacuum (+) until pumping feels slightly uncomfortable (not painful), then decrease (-) slightly.

VI. CLEANING INSTRUCTION

Wash: Before first use and after each use

Step 1-Rinse all separated parts that touch the breast and breast milk in cool water to remove breast milk residue

Step 2-Soak in warm soapy water for 5 minutes, then wash and rinse.

Step 3-Allow parts to air dry. Store dry parts in a cool place when not in use.

Do not store wet or damp parts.

Note:

- Take care not to damage parts of the breast pump kit during cleaning.
- Store the dry breast pump kit in a clean bag or container until next use.
- Wash hands thoroughly.

LONG-TERM STORAGE

Your breast pump contains a rechargeable battery. To preserve your breast pump, when not in use for an extended amount of time, it will automatically transition into storage mode. Before storing your breast pump for an extended amount of time, remember to:

- Clean breast pump kit • Clean Breast Pump • Make sure breast pump battery is fully charged

TROUBLESHOOTING

If breast pump does not power on

- Check that the breast pump is plugged into a working electric outlet
- Check the USB cable is working for other device.

If there is low or no suction

- Check that all of the breast pump and kit connections are secure.
- Disassemble and assemble the pump set as directed in the How to Use: Assembly, make sure all pieces are clean and not damaged.
- While pumping, make sure the breast petal form a complete seal around the breast.

If there is a breast milk overflow

- Turn off the breast pump and unplug the USB cable from the electrical outlet.
- Use a damp (not wet) cloth to wipe the breast pump.
- If there is milk in the tube: Remove tube from the pump unit as well as breast petal assembly
- When all components are dry, reassemble

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