Cat. No 407

GET READY!

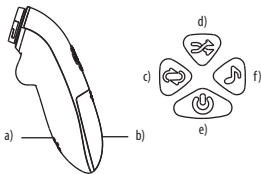
babypop

GB	Electronic nasal aspirator	2
PL	Elektroniczny aspirator do nosa	4
BG	Електрически аспиратор за нос	6
CZ	Elektrická nosní odsávačka	8
DE	Elektrischer Nasensauger	10
ES	Aspirador nasal eléctrico	12
ET	Elektriline ninaaspiraator	14
FR	Aspirateur nasal électrique	16
GR	Ηλεκτρική συσκευή ρινικής απόφραξης	18
HR	Električni nosni aspirator	20
HU	Elektromos orrszívó	22
LT	Elektrinis nosies aspiratorius	24
LV	Elektriskais deguna aspirators	26
NL	Electrische neusaspirator	28
PT	Aspirador nasal eléctrico	30
RO	Aspirator nazal electric	32
RU	Аспиратор назальный электрический	34
SE	Elektrisk nässug	36
SK	Elektrická nosová odsávačka	38
UA	Аспіратор назальний електричний	40
XS	Električni aspirator za nos	42

Electronic nasal aspirator

1. NAMES OF RESPECTIVE COMPONENTS

(1) Main unit (console)

- a) loudspeaker
- b) battery housing
- c) auto mode
- d) manual mode
- e) switch on/off
- f) melody mode

Manual mode: When using, always press and hold the manual mode button. If you release the button, the main unit (console) will switch off. Manual mode has only one level of suction.

Auto mode: has three suction intensity levels. To select another level press the (c) button.

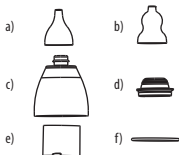
Melody mode: Change/select the melody.

Switch on/off: The aspirator ON/OFF switch.

Loudspeaker: Plays the selected melodies.

Battery housing: Shields the battery

(2) Aspirator


- a) Suction tip: Tip for removing nasal mucus secretion
- b) Suction tip: Tip for removing nasal mucus fluid
- c) Body of the aspirator: Housing protecting the interior of the aspirator
- d) Mucus chamber plug: prevents the reverse flow of the sucked out mucus secretion
- e) Mucus chamber: a container for collecting the sucked out mucus secretion
- f) Round gasket: A gasket between the body and the main unit (console)

2. PREPARATION BEFORE USE

1. Having carefully taken the device out of the packaging, make sure that:
 - the device is not damaged
 - no parts are missing

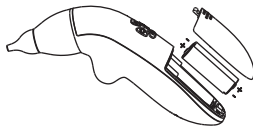- if the device is damaged or if parts are missing, contact the seller.
2. Before use
 - To operate, the device needs two 1.5V AA batteries (The batteries are not included with the device).
 - The device best operates on alkaline batteries.
 - Open the battery housing and properly insert the batteries bearing in mind their polarities (+/-) Close the battery housing

HOW TO USE THE BATTERIES SAFELY

- 1) Keep the batteries away from the reach of children
- 2) If batteries of different types are used, or if they are improperly inserted or if an old and a new battery are used together, liquid electrolyte may leak out.
 - Never use an old and a new battery together.
 - Never use an alkaline battery, a standard battery (zinc-carbon battery) or a rechargeable NiCD batter together.
- 3) In the event of a leak of electrolyte, immediately remove the batteries.
 - Leaking electrolyte may cause skin burns or other injuries. It may also damage the aspirator.
 - Batteries shall be disposed of in accordance with the applicable local regulations on the disposal of hazardous materials.
- 4) If the device is to be stored and will not be used for a longer time period, the batteries must be taken out.
- 4) If the device is to be stored and will not be used for a time period longer than two weeks, the batteries must be taken out.
- 5) The batteries must be replaced with new ones if the sound of the working pump or the sound of melodies fade down or if the device does not work with adequate efficiency.

3. HOW TO USE THE ASPIRATOR

- (1) Aspirator assembly
 - The mucus chamber should be closed with the plug.
 - The assembled chamber should be inserted into the device and twisted.
 - The suction tip should be attached to the body of the aspirator.
 - 2) The aspirator should be placed in the main unit (console)
 - the aspirator assembled as above should be placed in the main unit (console) port.
 - turn as indicated by the arrow
 - 3) Switch the power on.
 - Manual mode: In this mode of operation, press and hold the manual mode switch (d).
- If you release the button, the main unit (console) will switch off. Manual mode has only one level of suction.
- Auto mode: There are three suction intensity levels. To select another level press the button.
 - Melody mode: If necessary, you can help your child relax by playing a

selected melody. Every queued pushing of the melody mode button starts a different melody. (one of three). When one melody ends, another starts. To stop a melody, press the melody mode button again.

- Before switching the aspirator on, make sure it is attached to the main unit (console).
- The device is NOT a toy. Children MUST NOT play with the device. Each component of the device can be swallowed by a child and the child can choke.

4. SPECIFICATIONS

- Name of the device: Electric nasal aspirator
- Model number: XBQ-A01
- Rated voltage 3V
- Rated electric current: 500mA
- Rated power: 1.5W
- Battery: Battery 1.5V AA, 2 pieces
- Intended use: A device for removing nasal mucus secretion via a vacuum pump.

5. AFTER USE

- 1) Disassemble the aspirator and detach it from the main unit (console).
- 2) Remove the secretion collected in the mucus chamber
- 3) Sterilise the disassembled components of the aspirator by washing them in boiling water or precisely clean them under running water.
- 4) Dry carefully after washing.
- When sterilising the aspirator's components in boiling water, prevent them from touching the hot bottom of the water container.
- 5) Wipe the main unit (console) with a soft cloth.
- The main unit (console) must NOT be immersed in water or any other liquids.

6. STORAGE AND MAINTENANCE AFTER USE

- 1) Use or keep the device in a place where it is NOT exposed to contact with liquids.
- 2) Keep the device in a place where it is NOT exposed to high moisture, temperature, intensive sunshine, poor ventilation, shocks or vibrations.
- 3) In the event of mechanical damage to the device, do not try to disassemble the device but contact customer service.
- 4) Regularly inspect the condition of the device components.

7. SAFETY MEASURES

- You are kindly requested to comply with the "Safety recommendations", which describe proper and safe use of the device and protect you from any hazards.
- The "Safety recommendations" are marked as "Warning" or "Caution" in accordance with the description below.

WARNING A potentially hazardous situation, which, if not prevented, can result in critical or serious damage.

CAUTION A potentially hazardous situation, which, if not prevented, can result in minor damage.

This symbol means you should take special care because there is potential damage in particular circumstances. You are requested to follow the recommendations - this way you will avoid any potential hazards.

- Persons suffering from the below listed diseases should consult a doctor before use.
- Persons suffering from heart diseases, malignant tumours and diseases requiring rest and relaxation

- Persons suffering from cardiac disorders, with sensitive skin or other skin diseases
- Persons suffering from psychological disorders, with scars on the nose, suffering from frequent swelling of the body or high fever
- Persons under regular medical care due to other health problems or persons who regularly use products in connection with present or past health problems
- Persons under medical care due to health problems or allergies concerning the nose, the eyes and the ears or persons who require urgent medical intervention
- The device must NOT be used for any purposes contrary to its intended purpose.

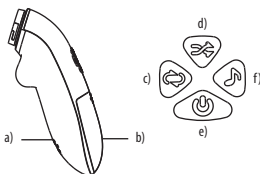
CAUTION:

1. Avoid the overflowing of the suction pump chamber. It must always be cleaned after use.
2. Switch the power off before detaching the aspirator from the body of the device.
3. Having checked the switch status, make sure the device operates properly.
4. When using the device, pay particular attention to the following issues:
 - 1) Check the condition of the device and check if its use is not uncomfortable for the patient.
 - 2) If the patients feels uncomfortable with the device use or if the device does not operate properly, switch the aspirator off to make sure the patient is safe, then undertake further steps.
 5. Before using the device, read the instruction manual carefully.
 6. During the device storage period, pay particular attention to the following issues:
 - 1) Keep the device away from the reach of children
 - 2) Always keep and use the device in dry environment
 - 3) Always keep the device in a place free of any adverse effects of its storage conditions, such as polluted air, adverse temperatures, moisture, ventilation, dusting, wind, presence of salts or other pollutants.
 - 4) Pay attention to the sloping surface, vibrations or shocks.
 - 5) The device must not be stored near any chemicals or gases.
 - 6) The device must not be stored near any sources of heat e.g. fireplaces, radiators, stoves, tv sets or other household appliances.
 - 7) The device must be stored in cleaned and ready to use condition.
 7. If you find any fault in the device operation, do not try to disassemble the device but contact customer service.
 8. NEVER modify the device in any way.
 9. Maintenance
 - 1) Regularly inspect the condition of the device and its components.
 - 2) Avoid continuous use of the device for a longer time period and make sure the device operates as usual if prior stored.
 - Never allow children to play with the device. Some of the device components can be swallowed, which can lead to choking.
 - Consult a doctor regarding the manner and frequency of use of the device on children.
 - NEVER insert the device too deeply into the nose. The tips of the device must NEVER be pressed too hard to the nostril walls.
 - NEVER insert the device into the ears, mouth or eyes. The aspirator that is described here is intended for the removal of mucus secretions from the nose. • The device must NOT be used for any purposes non-compliant with its intended purpose.
 - In case of use of the baby cleaning tweezers, use them with protective tips or otherwise baby's nose can get injured.

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