Troquel
120x120 mm

Área libre de texto
(Mínimo 5 mm)

Área de texto

Troquel
(no imprimible)

Anverso

120 mm

120 mm

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INSTRUCTIONS FOR USE OF NASO LIBERO INHALATION 3% + HA

INTENDED USE

NASO LIBERO Inhalation 3 % + HA, in single dosage format, without preservatives, for nebulization, acts via various mechanisms that facilitate the movement of respiratory tract secretions, thereby improving breathing by:

- Breaking the ionic bonds in the mucus gel, thus reducing the secretion's level of viscosity and elasticity.
- Absorbing water from the mucosa and sub-mucosa and thereby reducing swelling from the respiratory tract walls.
- Inducing cough and sputum, which may help eliminate mucus from the bronchi, and thereby improve obstruction of the respiratory tract.

NASO LIBERO Inhalation 3 % + HA in inhalation therapy is especially indicated for all age patients suffering from bronchiolitis, acute bronchitis, cystic fibrosis and chronic obstructive pulmonary disease (COPD). It can also be used as vehicle of other drugs products for inhalation.

USAGE INSTRUCTIONS

It is recommended for use in pneumatic nebulizers where optimal operating results have been obtained. It is recommended to follow the usage instructions of the equipment manufacturer.

Use 4 or 5 ml of the solution in the inhalation therapy according to medical prescription.

5 mm

30 mm

30 mm

5 mm

Reverso

120 mm

120 mm

PRECAUTIONS

- Keep out of reach of children.
- This product has no preservatives. After opening the ampoule used immediately and discard after use to avoid risk of contamination and being any cause of infection.
- Do not use jointly with other drugs unless first checking with your physician.
- In very sensitive patients, administration should be carried out according to medical prescription, since the prior administration of a bronchodilator may be necessary.
- In the case of bronchospasms or persistent cough, stop treatment and check with your physician.
- Do not use the product after the expiry date.
- Do not use if the ampoule is damaged.
- Do not store above 25°C.

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Distributor for Bulgaria: Linea Medica Ltd.