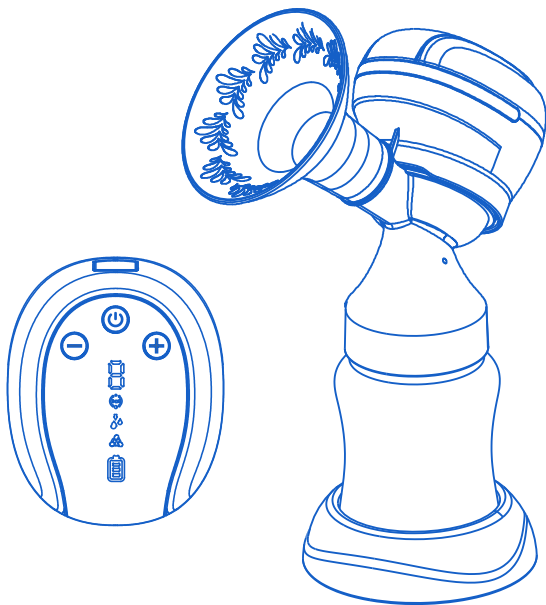
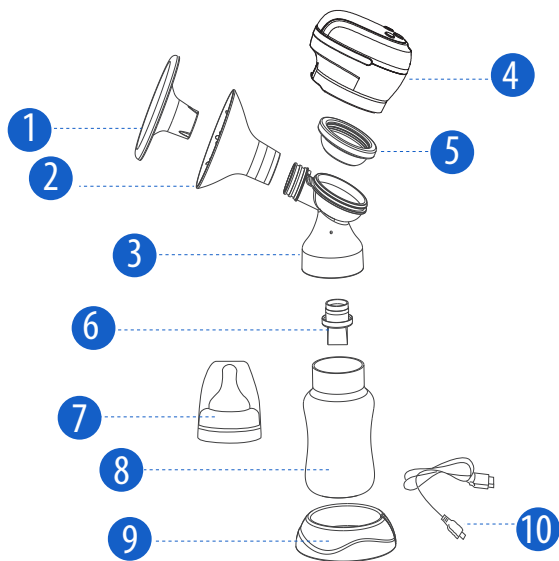
# LEIA PLUS

*integrated LED breast pump*

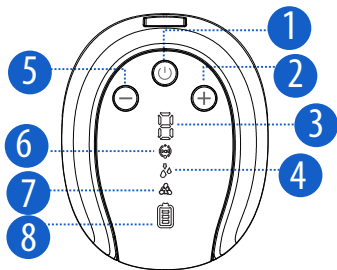
## INSTRUCTIONS FOR USE

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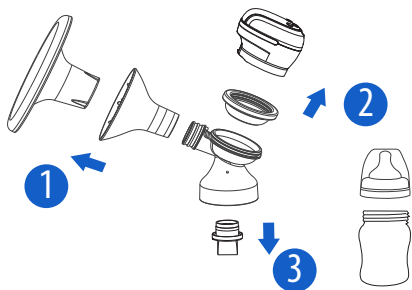
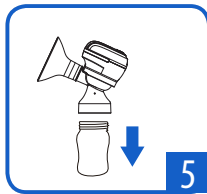
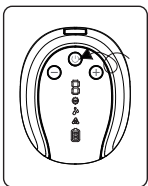
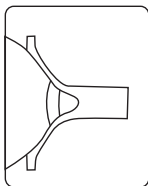
IV.

V.

When to Wash	Breast Shield 	Breast Pump Kit 	Bottle & Cap 
Before 1st use	<ul style="list-style-type: none"> <li>☞ wash</li> <li>☞ sanitize</li> </ul>	<ul style="list-style-type: none"> <li>☞ wash</li> <li>☞ sanitize</li> </ul>	<ul style="list-style-type: none"> <li>☞ wash</li> <li>☞ sanitize</li> </ul>
After each use	<ul style="list-style-type: none"> <li>☞ wash</li> </ul>	<ul style="list-style-type: none"> <li>☞ wash</li> </ul>	<ul style="list-style-type: none"> <li>☞ wash</li> </ul>
Once per day	<ul style="list-style-type: none"> <li>☞ sanitize</li> </ul>	<ul style="list-style-type: none"> <li>☞ sanitize</li> </ul>	<ul style="list-style-type: none"> <li>☞ sanitize</li> </ul>

VI.

Freshly Expressed Breast Milk Storage Guidelines (for Healthy Term Babies) <sup>1</sup>			
<sup>2</sup> Room Temperature	<sup>3</sup> Cooler With Ice Pack	<sup>4</sup> Refrigerator	<sup>5</sup> Freezer
<sup>6</sup> 4-6 hours at 66-78°F (19-26°C)	<sup>7</sup> 24 hours at 59°F (15°C)	<sup>8</sup> 3-8 days at 39°F or lower (4°C)	<sup>9</sup> 6-12 months 0 - -4°F (-18- -20°C)

VII.

## INTRODUCTION

Congratulations on the birth of your baby! You will soon find that breastfeeding your baby is a worthwhile experience. Breast milk will provide your baby with the best nutrition for healthy development.

This electric breast pump can simulate the baby's sucking actions to provide comfortable and pain-free service for breastfeeding mothers. FOR PROPER AND SAFE USE OF THE PUMP, PLEASE READ THE INSTRUCTIONS CAREFULLY AND PLACE IT TO BE EASILY ACCESSIBLE FOR FUTURE REFERENCE.

## PRODUCT FEATURES

• 9 Massage levels + 9 Pumping levels + 9 levels of Bionic

3-Mode Expression Technology: Massage/Pumping/Bionic

5-Mode Smart Functionality: Massage/Pumping/Bionic/Automatic/Memory

• Anti-back Flow Design for Hygiene

Closed system to prevent breastmilk from backing up to the pump, prevent infection, and keep the milk fresh.

• Super Soft Silicone Breast Shield

The flexible, ultra-soft silicone Breast Shield feels comfortable, reduces pressure and helps massage the breast to promote letdown and optimize milk flow.

• Quiet and Lightweight

The quiet breast pump allows for discreet pumping anywhere. Lightweight, portable pump. Designed for moms on the go who pump several times a day.

• Easy to Clean and Assemble

The pump has a mere five parts to assemble and clean, making it very easy to use.

• Smart Technology

Convenient pumping experience. Digital display features a memory function and a timer for easy tracking and recording pumping sessions.

• USB Rechargeable Battery

Durable lithium battery with 2-hour battery life. Micro USB port with standard USB cable for connecting to power source.

• Wireless Design

The pump has been designed with style in mind. No tubes, no wires – easy to use anywhere.

• BPA-free Compatible Pump

Pump and accessories are made from BPA-free materials. Pump is compatible with most standard baby bottles.

## I. PRODUCT COMPONENTS

1. Dust Cap; 2. Silicone Shield; 3. Connector; 4. Pump Machine; 5. Silicone diaphragm; 6. Valve; 7. Milk Bottle; 8. Bottle Base;

9. Nipple set; 10. Charging Cable;

Technical Specification: Power in:110-240V 50-60Hz Power Out:5V 1A; Battery Capacity and Type:3.7V 1400mAh Li-ion

## II. DISPLAY SPECIFICATION

1. Mode change button; 2. Suction Increase; 3. Suction Level; 4. Pumping; 5. Suction decrease; 6. Massage Mode; 7. Bionic mode; 8. Battery percent;

Power Button: Press 2 seconds to start or off the breast pump.

Mode Change: Touch the screen to change mode, do not touch long time. The initial stimulation mode will automatically change to expression mode after 2 minutes of use. Digital display features a memory function and timer for easy tracking and recording pumping sessions. When switched on, the pump's memory will automatically use the suction settings from the previous pumping session. The pump will automatically switch off after 30 minutes use.

Suction Level: + suction increase - suction decrease

Battery Percent: it shows the rest battery percent, please charge when is low battery

Session Duration: It shows the time you have used for the mode you chose; the machine will stop every 30 minutes.

Note: Long Press + button 2 seconds to turn off the light.

## III. Getting started

1. Remove the cap from the breast shield before each use.

2. Separate the connector from the pump.

3. Pull down on valve to remove from connector.

Before first use, charge the breast pump battery for 2 hours by connecting the pump to a power source with the USB connector. The battery icon will appear fully charged on the display when charging is complete.

## IV. ASSEMBLING YOUR BREAST PUMP

1. Wash hands thoroughly with soap and water before touching breast pump, breast pump kit and breast.

2. Press the suction bowl and pump machine into the connector.

3. Push the silicone shield onto the connector body.

4. Put the valve into the connector body.

5. Lock the milk bottle into the connector by rotating clockwise until tight.

- Check breast pump kit components for wear or damage before use. Replace if necessary.
- Always inspect all parts prior to use for cleanliness.
- To prevent damage to the breast pump all components must be completely dry before use.

#### V. OPERATING YOUR BREAST PUMP

Step 1- Place the breast shields on your breasts so that your nipples are properly centered in the tunnels.

Step 2- Hold the breast shield and connector onto your breast with your thumb and index finger. Support your breast with the palm of your hand.

Step 3- Turn on the breast pump by pressing the power button and choose the mode you prefer.

Step 4- Turn off the breast pump by pressing the power button after using and make the dust cap covered.

Notes:

1. Your breast pump will begin in Massage Mode. Adjust the pumping level using the (-)(+) controls to find a comfortable level.

Massage Mode is fast sucking pump rhythm to stimulate milk flow.

2. After two minutes, your pump will automatically transition to Pumping Mode. Adjust the pumping level using the (-)(+) controls to find your maximum comfortable vacuum. If your milk begins to flow before two minutes, press the mode change button POWER to manually transition to Pumping Mode.

Pumping Mode - slower sucking/pumping rhythm for gentle and efficient milk removal after milk has started flowing.

3. Once you are pumping in the Pumping Mode, increase the speed/vacuum (+) until pumping feels slightly uncomfortable (not painful), then decrease (-) slightly.

4. You can switch to the Bionic Mode when the milk flow stops. Bionic Mode aims to trigger another let-down by combining fast and slow pumping rhythms.

#### VI. CLEANING INSTRUCTION

Wash: Before first use and after each use

Step 1- Rinse all separated parts that touch the breast and breast milk in cool water to remove breast milk residue

Step 2- Soak in warm soapy water for 5 minutes, then wash and rinse.

Step 3- Allow parts to air dry. Store dry parts in a cool place when not in use.

Do not store wet or damp parts.

Note:

- Take care not to damage parts of the breast pump kit during cleaning.
- Store the dry breast pump kit in a clean bag or container until next use.
- Wash hands thoroughly.

#### LONG-TERM STORAGE

Your breast pump contains a rechargeable battery. To preserve your breast pump, when not in use for an extended amount of time, it will automatically transition into storage mode. Before storing your breast pump for an extended amount of time, remember to:

- Clean breast pump kit • Clean Breast Pump • Make sure breast pump battery is fully charged

#### TROUBLESHOOTING

If breast pump does not power on

- Check that the breast pump is plugged into a working electric outlet
- Check the USB cable is working for other device.

If there is low or no suction

- Check that all of the breast pump and kit connections are secure.
- Disassemble and assemble the pump set as directed in the How to Use: Assembly, make sure app pieces are clean and not be damaged.
- While pumping, make sure the breast petal form a complete seal around the breast.

If there is a breast milk overflow

- Turn off the breast pump and unplug the USB cable from the electrical outlet.
- Use a damp (not wet) cloth to wipe the breast pump.
- If there is milk in the tube: Remove tube from the pump unit as well as breast petal assembly
- When all components are dry, reassemble

#### VII. FRESHLY EXPRESSED BREASTMILK STORAGE GUIDELINES (FOR HEALTHY TERM BABIES).

2. Room temperature; 3. Cooler with ice pack; 4. Refrigerator; 5. Freezer; 6. 4-6 hours at 19-26°C (66-78°F); 7. 24 hours at 15°C (59°F); 8. 3-8 days at 4°C (39°F); 9. 6-12 months at 0- -40°C (-18- -200°F);

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