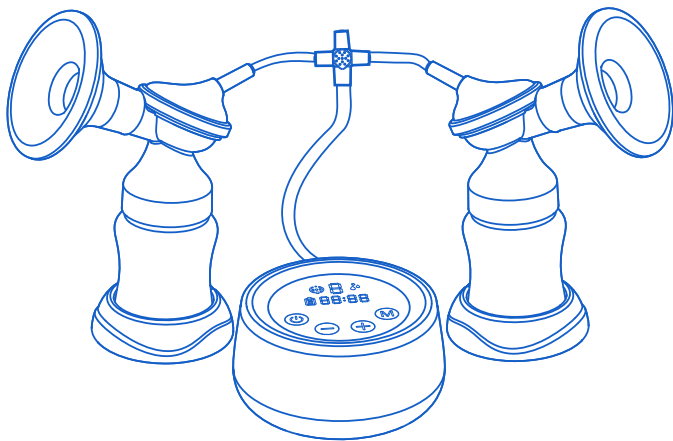
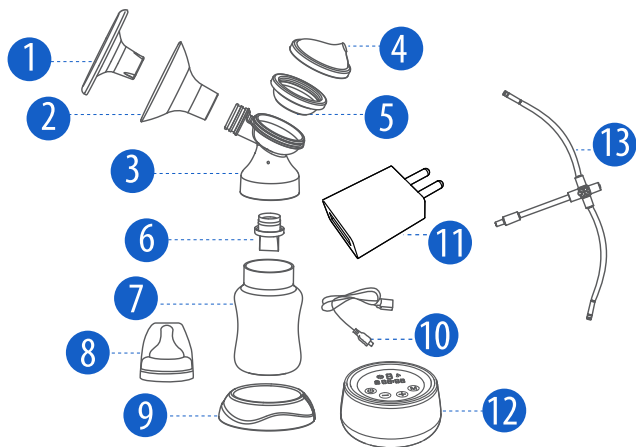
NESSA

double electric breast pump

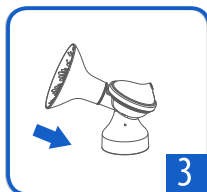
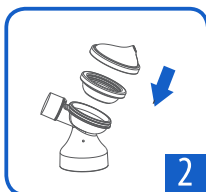
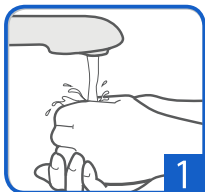
INSTRUCTIONS FOR USE

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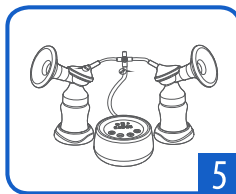
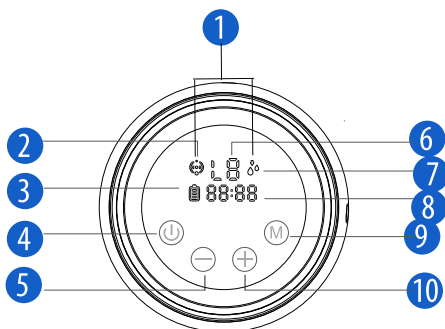
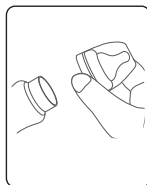
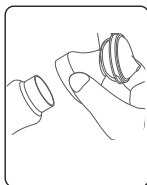
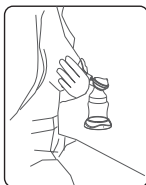
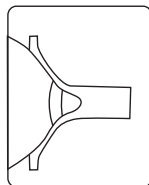
III.

IV.

When to Wash	Breast Shield 	Breast Pump Kit 	Bottle & Cap 
Before 1st use	☞ wash ☞ sanitize	☞ wash ☞ sanitize	☞ wash ☞ sanitize
After each use	☞ wash	☞ wash	☞ wash
Once per day	☞ sanitize	☞ sanitize	☞ sanitize

V.

A

B

C

VI.

Freshly Expressed Breast Milk Storage Guidelines (for Healthy Term Babies)

Room Temperature ²	Cooler With Ice Pack ³	Refrigerator ⁴	Freezer ⁵
4-6 hours ⁶ at 66-78°F (19-26°C)	24 hours ⁷ at 59°F (15°C)	3-8 days ⁸ at 39°F or lower (4°C)	6-12 months ⁹ 0 - -4°F (-18 - -20°C)

VII.

Thank you for buying our product. Please read this manual carefully before using and keep it for easily checking in the future. Wish you have a pleasant using experience with our smart electric breast pump.

PRODUCT FEATURES

- BPA-free materials & Compatible: Pump and accessories are made of BPA-free materials. Connector is compatible with most standard baby bottles.
- Anti-back flow protection design: Closed system to prevent breastmilk from backing up to the pump, prevent infection, and keep the milk fresh.
- Super soft silicone breast shield: The flexible, ultra-soft silicone breast shield feels comfortable, reduces pressure and helps massage the breast to promote letdown and optimize milk flow.
- Smart touch screen technology: Digital display features a memory button, timer and backlight for easy tracking and recording pump sessions.
- Meet different needs: 9 levels Massage mode & 9 levels Pumping mode & 9 Levels Bionic mode.
- Quiet and Lightweight: The quiet breast pump allows for discreet pumping anywhere. Lightweight, portable pump designed for moms on the go who pump several times a day.
- Large battery capacity: Uses a large capacity rechargeable lithium battery, fully charged can be used for 2 hours, no need to connect to AC power. Moms can enjoy pumping anywhere at anytime.

I. PRODUCT COMPONENTS

1. Dust cap (x2); 2. Silicone shield (x2); 3. Connector (x2); 4. Diaphragm cap (x2); 5. Silicone diaphragm (x2); 6. Silicone valve (x4); 7. Milk bottle (x2); 8. Nipple set (x2); 9. Bottle base (x2); 10. Charging cable; 11. USB adaptor; 12. Main unit; 13. Airtube
 Technical Specification: Power in: 110-240V 50-60Hz; Power Out: 5V 1A; Size: 105*105*53mm; Weight: 275g; Battery capacity and type: 3.7V 1400mAh Li-ion.

II. ASSEMBLING YOUR BREAST PUMP

1. Wash hands thoroughly with soap and water before touching breast pump, breast pump kit and breast.
 2. Press the silicone diaphragm and cap into the connector.
 3. Push the breastshield onto the connector.
 4. Put the valve into the connector. Lock the milk bottle into the connector by rotating clockwise until tight.
 5. Push the tube into the cap/s and main unit. Push two tubes if you choose double side pumping. Tap the screen to turn on the main unit.
- A- Single side pumping; B- Double side pumping;
- Check breast pump kit components for wear or damage before use. Replace if necessary.
 - Always inspect all parts prior to use for cleanliness.
 - To prevent damage to the breast pump all components must be completely dry before use.

III. ICON DISPLAY

1. Bionic mode; 2. Massage mode 3. Battery Indicator; 4. Power button; 5. Suction decrease; 6. Suction level; 7. Pumping mode; 8. Time indicator; 9. Mode change; 10. Suction increase;
 Power button Press 2 seconds to turn on or to turn off the breast pump.

1. Your breast pump will begin in Massage Mode. Adjust the pumping level using the (-)(+) buttons to find a comfortable level. Massage Mode is fast sucking pump rhythm, which stimulates milk flow.
 2. After two minutes, your pump will automatically transition to Pumping Mode. Adjust the pumping level using the (-)(+) buttons to find your maximum comfortable level of pumping. If your milk begins to flow before two minutes, press the Mode change button to manually transition to Pumping Mode.
- Pumping Mode is slower sucking/pumping rhythm for gentle and efficient milk removal after milk has started flowing.
3. Once you are pumping in the Pumping Mode, increase the speed/vacuum (+) until pumping feels slightly uncomfortable (not painful), then decrease slightly using the button (-).
 4. You can switch to the Bionic Mode when the milk flow stops. Bionic Mode aims to trigger another let-down by combining fast and slow pumping rhythms.
- Modes: Massage Mode L1-L9; Pumping Mode L1-L9; Bionic L1-L9.
- Battery Indicator: Indicates remaining battery life. Please charge breast pump when low in battery.
- Time Indicator: Indicates the length of your pumping session. The pump will automatically switch off after 30 minutes use.

IV. OPERATING YOUR BREAST PUMP

Step 1- Place the breast shield on your breast so that your nipple is properly centered in the tunnel.

Step 2- Hold the breast shield and connector onto your breast with your thumb and index finger. Support your breast with the palm of your hand.

Step 3- Turn on the breast pump by tapping the power button and choose the mode you prefer.

Step 4- Turn off the breast pump by pressing the power button. Remove the bottle from the connector. Install the nipple set if you are going to feed the baby right away or transfer the freshly expressed milk into a milk storage bag/container and store it in a freezer or refrigerator.

V. CLEANING INSTRUCTIONS

Wash before first and after each use!

Step 1- Disassemble all parts as shown in I. Remove the cap from the breast shield. Separate the connector from the breast shield, diaphragm, cap, valve and bottle. Rinse the Dust cap (I-1); Silicone shield (I-2); Connector (I-3); Silicone diaphragm (I-5); Valve (I-6); Nipple set (I-8); Milk bottle (I-7) and Bottle base (I-9) in cool water. NOTE: Never wash the Main unit (I-12); Charging cable (I-10), USB adapter (I-11) and Airtube (I-13) to avoid permanent damage to the product. Wipe with a damp cloth (not wet) and dry immediately.

Step 2- Soak in warm water for 5 minutes, then wash and rinse.

Step 3- Allow parts to air dry. Store dry parts in a cool place when not in use.

Do not store wet or damp parts

WHEN TO WASH AND SANITIZE:

- BEFORE FIRST USE WASH AND SANITIZE: the Breast shield (I-2); Breast pump kit (I-3; I-5; I-6); Nipple set (I-8) and Milk bottle (I-7).
- AFTER EACH USE WASH: the Breast shield (I-2); Breast pump kit (I-3; I-5; I-6); Nipple set (I-8) and Milk bottle (I-7).
- ONCE PER DAY SANITIZE: the Breast shield (I-2); Breast pump kit (I-3; I-5; I-6); Nipple set (I-8) and Milk bottle (I-7).

Note:

- Before first use, charge the breast pump battery for 2 hours by connecting the pump to a power source with the USB connector. The battery icon will appear fully charged on the display when charging is complete.
- Take care not to damage parts of the breast pump kit during cleaning.
- Store the dry breast pump kit in a clean bag or container until next use.
- Wash hands thoroughly.

VI. LONG-TERM STORAGE

Your breast pump contains a rechargeable battery. To preserve your breast pump, when not in use for an extended amount of time, it will automatically transition into storage mode. Before storing your breast pump for an extended amount of time, remember to:

A - Clean breast pump kit B - Clean the breast pump C - Make sure breast pump battery is fully charged.

TROUBLESHOOTING

If breast pump does not power on:

- Check whether the breast pump is plugged into a working electric outlet.
- Check whether the USB cable is working, using another device.

If there is low or no suction:

- Check whether all breast pump connections are secure.
- Disassemble and assemble the pump set. Make sure all pieces are clean and not damaged.
- While pumping, make sure the breast shield is completely sealed around your breast.

If there is a breast milk overflow:

- Turn off the breast pump and unplug the USB cable from the electrical outlet.
- Use a damp (not wet) cloth to wipe the breast pump.
- When all components are dry, reassemble.

VII. FRESHLY EXPRESSED BREASTMILK STORAGE GUIDELINES (FOR HEALTHY TERM BABIES).

2. Room temperature; 3. Cooler with ice pack; 4. Refrigerator; 5. Freezer; 6. 4-6 hours at 19-26 °C (66-78°F); 7. 24 hours at 15 °C (59°F); 8. 3-8 days at 4 °C (39°F); 9. 6-12 months at -18- -20 °C (0- -4°F).

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